Medical Equipment Roll Stand

Model:RS008-101-ZG

User Manual

CONTENT

1	Safety Information	2
2	Equipment Symbols	3
3	Area of application/ Intended use	3
4	Reporting obligation	3
5	CE Mark	4
6	Manual Safekeeping	4
7	Specification	5
8	Roll Stand Parts Schematic	6
9	Assemble the roll stand	7
10	Maintenance and Adjustment	9
11	Cleaning and disinfection	10

1 Safety Information

1.1 Dangers

There are no dangers that refer to the product in general. Specific “Danger” statements may be given in the respective sections of this operation manual.¹

1.2 Warnings

 **WARNING**

ℓ **Do not dismantle the wall frame by yourself.**

The maintenance of the roll stand must be carried out by maintenance personnel trained and authorized by the company.

ℓ **Handle packaging materials**

You must comply with local regulations or the hospital's waste disposal system. Packaging materials must be placed out of reach of children.

ℓ **MORTAL DANGER BY ELECTRIC SHOCK**

The electric cables of ME equipment and ME systems may only be connected to the power source and to the medical device respectively after finishing installation. Prior to maintenance or dis-assembly all power cables must be disconnected from the power source.

1.3 Cautions

 **CAUTION**

ℓ **MOUNTING THE MEDICAL ELECTRICAL EQUIPMENT**

Only when all screws are tightened and securely fixed can the ME equipment be mounted.

1.4 Note

Note

ℓ **Please select the corresponding medical equipment according to the model to install the monitor.**

2 Equipment Symbols

NOTE

ℓ Some symbols may not appear on all equipment.

	Attention: Consult accompanying documents (this manual).		Serial number
	Read Operator's Manual		CE mark
	Manufacturer		Importer
	Manufacture date		Medical Device

3 Area of application/ Intended use

Medical Equipment Roll Stand are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theaters as well as treatment rooms as auxiliary devices for doctors and medical staff.

4 Reporting obligation

Reporting obligation

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

5 CE Mark

6 Manual Safekeeping

For any questions that may arise at a later time please safe-keep our manual!

7 Specification

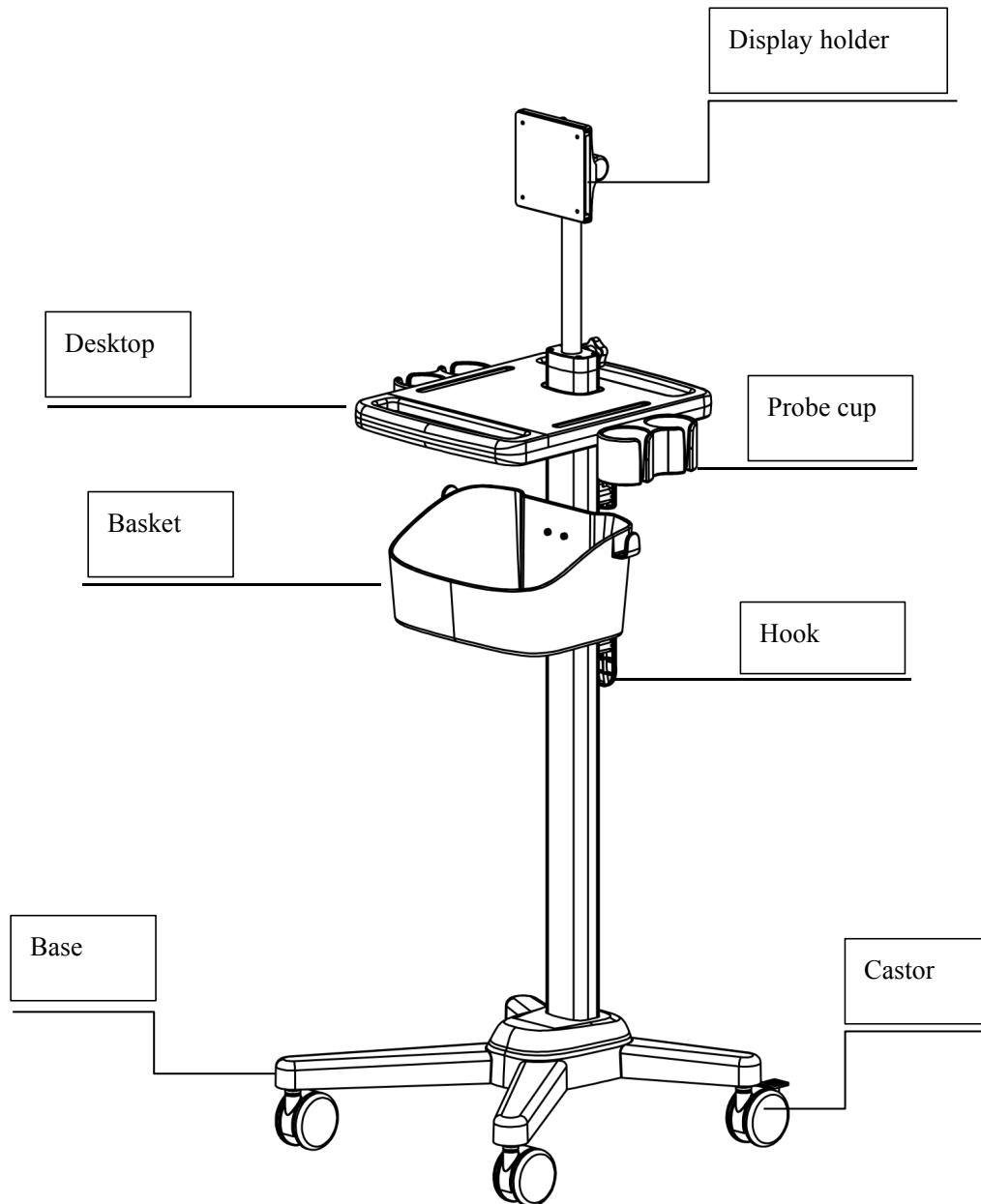
7.1 Hardware Specifications

Model	E.W.TB.RS008-101-ZG
Photo	
Load Capacity	1~8Kg as Standard
Trolley Size	450*469*1457mm
Height	1045~1457mm
Basket	300*150*213mm, <u>load 2kg</u>
Tablet	300*400mm, <u>load 2kg</u>
Adjust Way	Manual lift
Material	Aluminum alloy+Sheet metal+ABS
Package Size	<u>78*63*21.5cm (1PCS in 1 Carton)</u>
Net Weight	12±0.3kg (1PCS)
Gross Weight	15±0.3kg (1PCS)
Certificates	ISO13485/ISO9001/CE/FDA

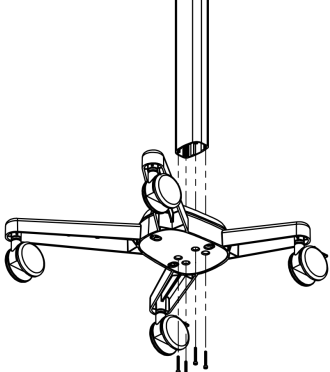
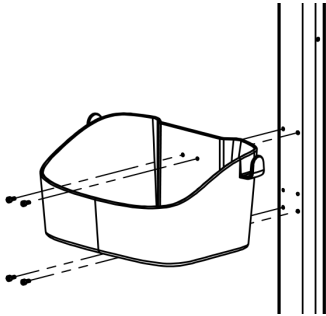
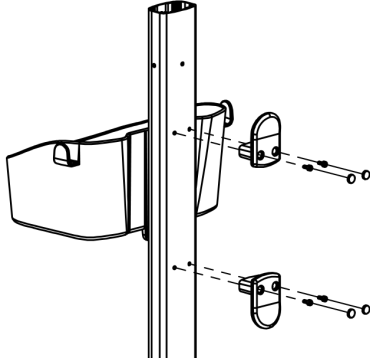
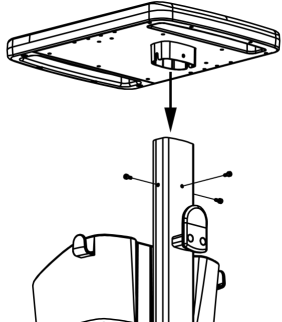
7.2 Environmental Specifications

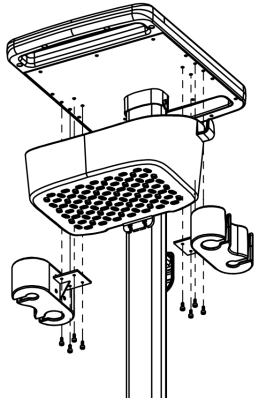
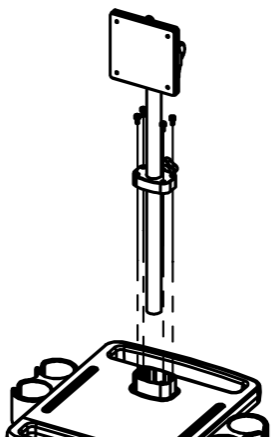
Temperature range	Operating	0 °C ~+40 °C
	Storage	-40°C~+70°C
Humidity range	Operating	15% - 95 %, non-condensing
	Storage	10% - 95 %, non-condensing
Altitude range	Operating	-500 ~ +4,600 m (57.0kPa~106.0kPa)
	Storage	-500 ~ + 4,600 m (50.0kPa~106.0kPa)

8 Roll Stand Parts Schematic

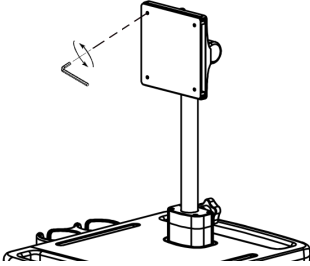
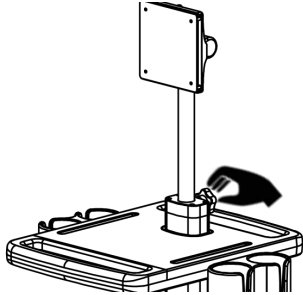
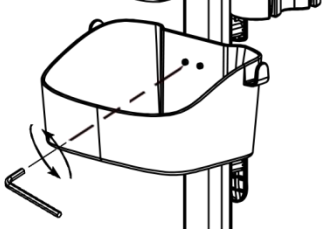
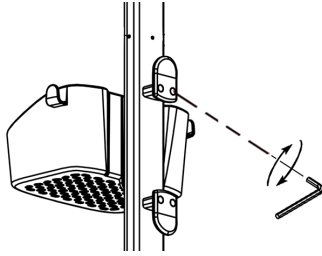
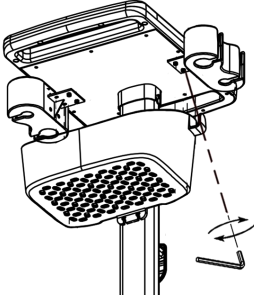
9 Assemble the roll stand

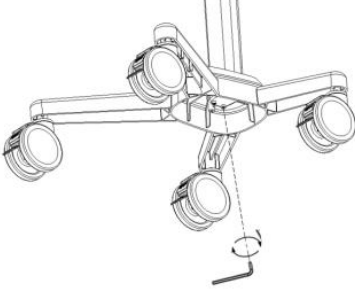
<p>1</p>	<p>Install the pole</p> <ul style="list-style-type: none">- Insert Roll Stand Pole to the base-Use 4 PCS hex screws and spacer to lock the pole.	
<p>2</p>	<p>Install the basket</p> <ul style="list-style-type: none">-Fix the basket to the pole mounting hole with 4 hexagon socket screws,Cover the screw holes with rubber plugs	
<p>3</p>	<p>Install the hook</p> <ul style="list-style-type: none">-Fix the hook to the pole mounting hole with 4 hexagonal socket screw,Cover the screw holes with rubber plugs	
<p>4</p>	<p>Install the desktop</p> <ul style="list-style-type: none">-Fix the desktop to the pole mounting hole with 3 grub screws	

<p>5</p>	<p>Install the Probe cup</p> <p>-Fix the Probe cup to the desktop mounting hole with 8 hexagonal socket screw</p>	
<p>6</p>	<p>Install the monitor stand</p> <p>-Fix the monitor stand to the pole mounting hole with 4 hexagonal socket screw</p>	

10 Maintenance and Adjustment.

Very little maintenance is required during the life span of the roll stand, however it is recommended to check all pivot /swivel points as well as all bearing points regularly . Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

No.	Items	Reference photo	Cycle
1	Display connection fixing screw		12month
2	Display lift adjustment screw		12month
3	Basket Screw		12month
4	Hook Screw		12month
5	Probe cup Screw		12month

	Pole Screw		12month
---	------------	--	---------

11 Cleaning and disinfection

Roll stand may be cleaned with mild , non-abrasive solutions commonly used in hospital environment (e.g. diluted bleach , ammonia , or alcohol solutions).

The surface finish may be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.

Do not use steel wool or other abrasive materials. Avoid all liquids entering the roll stand.

CAUTION: We make no claims concerning the efficacy of the disinfection process. Please consult your hospital's infection control manager

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