## Ready-to-Use Hydrophilic Intermittent Catheters

EN

### Instructions for Use

#### INDICATIONS FOR USE

The catheters are indicated for intermittent urinary catheterization by inserting through the urethra to pass urine from the bladder.

#### CONTRAINDICATIONS

- Acute urethritis
- Acute prostatitis
- Acute epididymitis
- Patients with calcareous urolithiasis

#### WARNINGS

- After infiltrating the catheter, do not wipe or scrub the surface of the catheter by either physical force or chemicals to avoid damaging the hydrophilic coating which may result in efficacy loss.
- This is a single use device. Do not re-sterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

#### PRECAUTIONS


- Federal law restricts this device to sale by or on the order of a physician.
- This product can be used in healthcare centers under supervision of appropriately qualified and trained personnel or at home by patients according to the instructions for use.
- This product is disposable, do not reuse.
- To prevent urinary tract infection, appropriate aseptic steps should be taken to areas around the urethral opening prior to insertion.
- Do not use if the package is damaged, contaminated, or expired.
- Special attention shall be taken for patients with disturbance of consciousness as involuntary pull of the catheter may lead to bladder and urethra mucosal injury or break the catheter.
- Intubation and extubation should be as gentle as possible to prevent urethral injury.
- Slowly advance the catheter through the urethra into the bladder. If substantial resistance is met, do not force the catheter. Slowly pull out the catheter and look for reasons.

#### ADVERSE REACTIONS

- Bleeding in the urethra
- Catheter-associated urinary tract infection
- Urethral injuries and inflammation

#### STEP-BY-STEP INSTRUCTIONS

1. Select an appropriate specification of catheter which is suitable for the specific procedure. Carefully inspect the package and catheter prior to use. Discard and replace the product if it is damaged. The operator and patient shall follow the standard aseptic procedure before conducting the catheterization.
2. Open the specification of products as below:  
**Ready-to-Use Hydrophilic Intermittent Catheter**  
Open the package and expose the funnel.
3. Holding the funnel, slide the contactless sleeve along the catheter towards the tip (Male).
4. Carefully insert the catheter through the urethral orifice until urine starts to flow out - this occurs after the eyelets of the catheter reach the bladder cavity.
5. For user positioning in front of a toilet facility, the urine can drain into the toilet. If lying down, place the funnel end of the catheter into a container or connect a drainage bag by inserting the plug of the drainage bag into the catheter funnel to collect the urine. Then hold the catheter until urine stops flowing.
6. When urine stops flowing, begin to withdraw the catheter. Stopping each time urine begins to flow until the bladder is empty.
7. After use, remove the catheter from the urethra and dispose of it in accordance with local administrative regulations to prevent environmental pollution and potential health hazards.
8. Wash hands after catheterization.

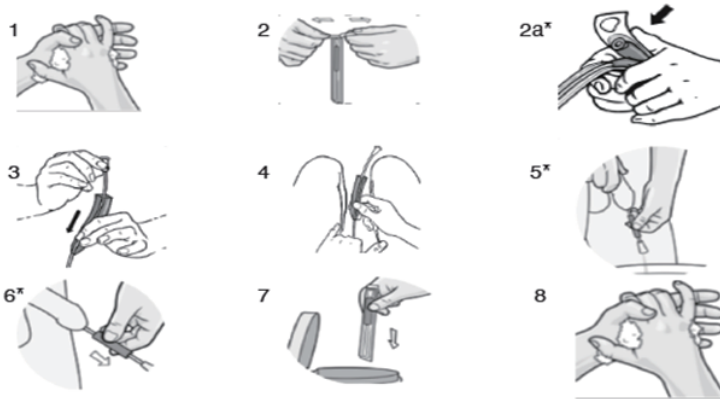\*Steps 5 & 6 are universal for both male and female users.

#### RECOMMENDED STORAGE CONDITIONS

- For single use only, do not repeat sterilization and reuse.
- Use immediately after the package is opened and destroy after use.
- The device should be stored in a cool, dry and dark place. Do not store catheters where they are directly exposed to organic solvents, ionizing radiation, ultraviolet light and high temperatures.

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